
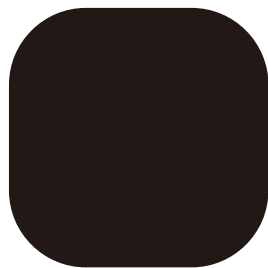


Auto-wrapping (tear-and-seal) portable toilet

wrappon · **SUNNY**

User's Manual

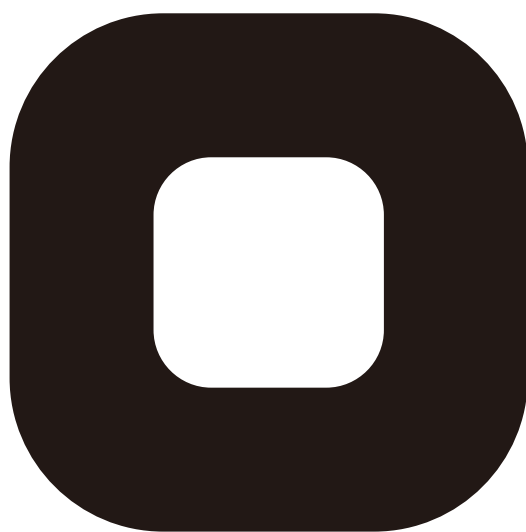

wrappon®

Auto-wrapping (tear-and-seal) portable toilet

Please read this manual very carefully and understand thoroughly before using it.
Please keep this manual clean.

 NIHON SAFETY CO.,LTD.

Introduction

How to Use

Store Properly

When this happens

Table of Contents

Introduction

Before using the Wrappon Sunny	3
Product Overview	3
About User's Manual	3
To Ensure Safe Use	4
Confirmation of Parts and Accessories and Introduction of Consumable Supplies	9
Name of various parts	10
External dimensions	10
Installed status	10
Top view with cover removed	10
Control panel and Display	11

Using the Wrappon Sunny

Main Unit Setup	12
Installing a Film roll	14
Changing the filmroll	15
How to use the toilet	16

Put it Away when it is not using

Maintaining the Product	18
Daily maintenance	18
If the wrapping unit becomes soiled	18
When installing a film roll	18
Storing the product	19
About the storage environment	19

When this happens

Frequently asked questions	20
Error Codes	21
Troubleshooting	22
Re-wrapping	23
How to take care of tangled film	24
Specifications	25
Wrappon Product Warranty & Free Replacement Policy	26

Before using the Wrappon Sunny

Thank you for purchasing Wrappon Sunny, auto-wrapping (tear-and-seal) portable toilet. This User's Manual explains how to correctly handle Wrappon Sunny as well as simple explanation and maintenance procedures. Please read this User's Manual thoroughly before using Wrappon Sunny for safe use.

Product Overview

- Wrappon Sunny is a clean, simple and odorless toilet that uses special coagulant and wraps the waste automatically.
- Please ensure that you use the designated film for operation.
- The products includes 2 sets of consumables for up to 30 uses each. Please purchase addition sets as needed.
- When assembling the product, be careful not to get your fingers caught in the main unit.
- For any questions on how to use the Wrappon Sunny, contact the dealer you purchased.


About the User's Manual

- Keep this User's Manual close at hand for easy references by users. Should this User's Manual be lost or damaged, purchase a new copy from the company you purchased.
- The manufacturer may modify this User's Manual at any time without prior notice.
- User's Manual and warning labels do not necessarily show the shapes of items and sometime those are the icons. Note that the display values shown in the diagrams are only examples.
- If you transfer this product to another party, be sure to transfer the User's Manual and the Quick Reference Guide.
- The intellectual property rights for this User's Manual belong to the manufacturer. No part of this User's Manual may be reproduced without written permission from the manufacturer.

To Ensure Safe Use

Before using the product, read the "To Ensure Safe Use" section thoroughly to ensure correct use. The precautions described here represent important safety-related issues and must be observed.

Precautionary symbols and its meanings		Various symbols used to indicate and explain the items that must be observed.	
 Warning	Using the product incorrectly and ignoring this symbol may cause death or serious injury		This type of symbol means "PROHIBITED". The one shown on the left means "Do not disassemble."
 Caution	Using the product incorrectly by ignoring this symbol may cause injury or property damage		This type of symbol means "REQUIRED:" Left sign means "MUST BE OBSERVED:"

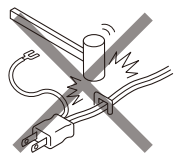 Warnings		
! MUST BE OBSERVED When using the AC adapter, connect it directly to an outlet(AC100V-240V)and securely insert the plug. If the adapter is not plugged in securely, electric shock or fire may result due to overheating.	! MUST BE OBSERVED Use only designated AC adapter. Use only the specified AC adapter. If not, it may damage the product or pose a danger.	 PROHIBITED  Do not use staples, nails, or wires to fasten the power supply cord. May damage the power supply cord causing electric shock, short circuit, or fire.
 DO NOT TOUCH WITH WET HAND Do not touch with wet hand . May cause electric shock if you touch the electric plug and switch with wet hand.	 DO NOT WET WITH WATER May cause short in the electrical circuit and corrosion on the product.	! MUST BE OBSERVED Reinforcement bar must be installed. Without it, the product may fall resulting injury.

⚠ Warnings

▲ PROHIBITED

Be careful not to damage the cords, plugs, etc.

Do not damage, do not modify, keep away from high temperature, do not forcibly bend, do not twist, do not pull, do not place under a heavy object, or do not bundle them.

- A damaged cord or plugs, if used, may cause electric shock, short circuit, or fire.
- When unplugging, hold the plug itself and pull out straight.
- If the power supply cord becomes damaged, ask the seller to repair it.

▲ PROHIBITED

When the main power supply is ON, do not put your hand in the film or toilet.

The rollers or seal bar may trap your hand causing severe injury.

▲ PROHIBITED

Do not insert hand into the opening area from the bottom of the product.

Inside the product, there are moving parts which may damage and burn your hands.

▲ DO NOT DISASSEMBLE

Do not attempt to disassemble or repair the product.

High voltages are present inside the product, which may cause electric shock, fire, or product malfunctioning result in injury.

▲ PROHIBITED

Use the product only as a toilet and do not use for other purposes.

- Do not place inside a vehicle and use it as a seat.
- Do not use the product as a step.
- Do not use the product as a pallet.
- Do not use the product as a chair while taking the toilet seat off.

The above may cause the product to topple or be damaged, causing injury.

! MUST BE OBSERVED

Regularly clean the dust off from the power supply plug.

Accumulated dust may cause an insulation failure, resulting in fire due to overheating. Disconnect the power supply plug.

! MUST BE OBSERVED

If you hear thunder, do not touch the main unit or power supply plug. May cause electric shock.

▲ PROHIBITED

Do not place a heavy load on the product.

- Do not stand on the product.
- Do not move the product while person is on the product.
- Do not use two persons at the same time.

! UNPLUG

Before maintenance or inspection, disconnect the power supply plug.

If not unplugged, you may get an electric shock or an injury.

To Ensure Safe Use (continued)

⚠ Warnings		
△ NO OPEN FLAME • Do not install the product near an open flame. • Do not throw flammable material or lit cigarettes into the toilet. Deform the product may cause short circuit or fire which may result in waste leakage. Do not use the product near a stove.	▲ PROHIBITED Children or adults who require assistance in moving must not use the product by themselves. • May result in injury if mischief movement is done by a child. • May result in falling down of the product.	▲ PROHIBITED Do not use the person heavier than 150kg. It might be damaged and may result in injury.
! MUST BE OBSERVED Do not put the coagulant into your mouth. The coagulant is not food. Check the precautions for use.	! MUST BE OBSERVED The product must be used on a flat and stable location. If installed in an unstable location, the product may fall down and cause injury.	! UNPLUG If the product is not to use for a long time, disconnect the power supply plug from the electrical outlet. Insulation deterioration may cause electric shock or fire.
△ Caution		
! MUST BE OBSERVED Before use, make sure that you check if the film roll is installed. If used without film roll installed, waste leakage may cause the product failure. Also no-load running may cause the product failure.	▲ PROHIBITED Do not throw large quantity of toilet paper into the toilet. When wrapping the film, the toilet paper may get tangled with the film, resulting bad seal with a leakage.	▲ PROHIBITED Do not install, use and store at the following places. • Lots of dust and dirt. • Places such as bathroom over 80% humidity. • Locations exposed to rain and direct sun light. If you install, use and store the product in these places, you may get the trouble.

△ Caution

▲ PROHIBITED

Do not put in more than the specified amount of coagulant.

The coagulant may become trapped between the rollers which may cause the product to fail.

For the coagulant information, please refer the coagulant note on use.

▲ PROHIBITED

Do not use anything except the designated coagulant.

Using any other coagulant may cause insufficient coagulation, clogging, wrapping failure, or product failure.

! MUST BE OBSERVED

The user must sit squarely on the toilet seat.

If the user sits close to the front edge, the waste may leak and cause the product to fail.

! MUST BE OBSERVED

Throw used toilet paper into the middle of the toilet.

If not observed, it may cause clogging and wrapping failures.

▲ PROHIBITED

Do not press the push button while seated on the toilet.

The film will get caught in the toilet seat, causing product failure or wrapping failure.

▲ PROHIBITED

Use the product only if wrapped film does not remain on the bottom cover.

If wrapped film remains, it may cause product failure or wrapping failure.

! MUST BE OBSERVED

When installing the film roll, the power supply cord must be disconnected.

If not observed, it may cause clogging and wrapping failures.

▲ PROHIBITED

Do not use other film roll than designated film roll.

Using other film roll may cause it to get trapped inside the product, and may cause waste leakage or product failure.

! MUST BE OBSERVED


Install the toilet seat to the body properly.

Wrong installation may cause the product fall or product malfunction. This may result in product failure.


! MUST BE OBSERVED

Make sure that you press the button for initializing after installing a new film roll.

If you do not run the initial film-bonding operation, the waste may leak, causing the product to fail as the film is not sealed.

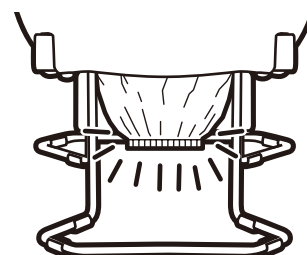


To Ensure Safe Use (continued)

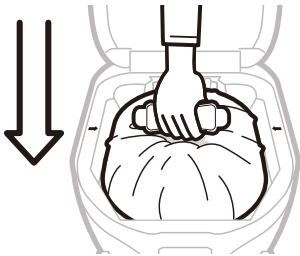⚠ 注意		
⚠ MUST BE OBSERVED Push down the film installation jig while aligning it with the protrusions on the top plate.  If the product is used without a film installation jig, film may be dislocated from rollers and unbalanced film feeding which may cause the wrapping failure.	▲ PROHIBITED To clean the product, use a diluted solution of mild kitchen detergent. Do not use any of the following : toilet cleaner , benzene, solvent, or cleanser. Using such a prohibited product may damage the product, causing injury. It may also cause product deformation or discoloration.	▲ PROHIBITED Do not move without wrapping. Waste may leak. The floor and the product is soiled. So make sure that you move the product after completing the wrap cycle.
▲ PROHIBITED Do not relocate the product while connecting the power source plug. You may trip on the cord resulting the disconnection of the plug or short circuit.		


The Parts and Accessories List and Description of Consumable Supplies

The Parts and Accessories List


- The parts and accessories shown below are pre-packaged with the product.
- Before using the product, check to make sure all of the accessories are intact. Should any accessory be missing or damaged, please notify the seller of the product immediately.

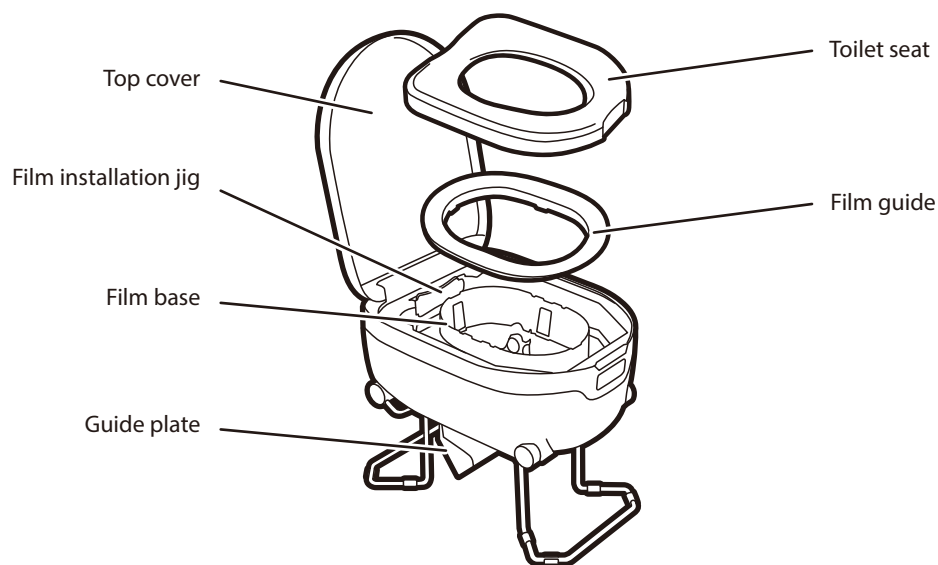


Accessories List

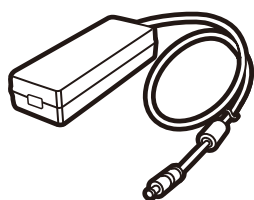

AC adapter (1)

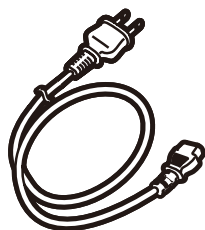

Power supply plug (1)

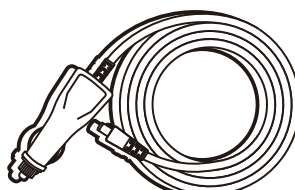


DC cable (1)



Transport bag (1)



Special coagulant
(30 × 2 packets/box)

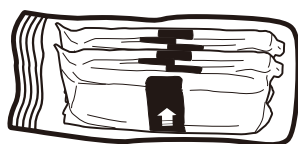

Film roll
(30 × 2 times per roll)



User's Manual (1)




Oil & Cloth (1)



* Helps if the film sticks after long use.
See P.15-5 for instructions

Name of various parts


External dimentions(mm)

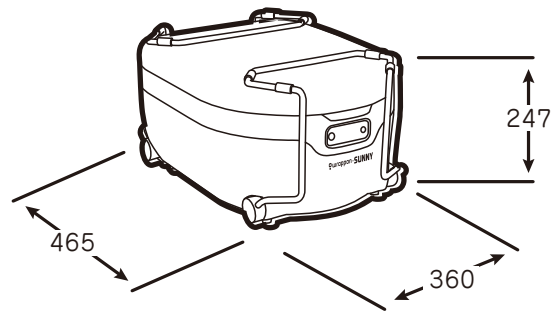


Installed status

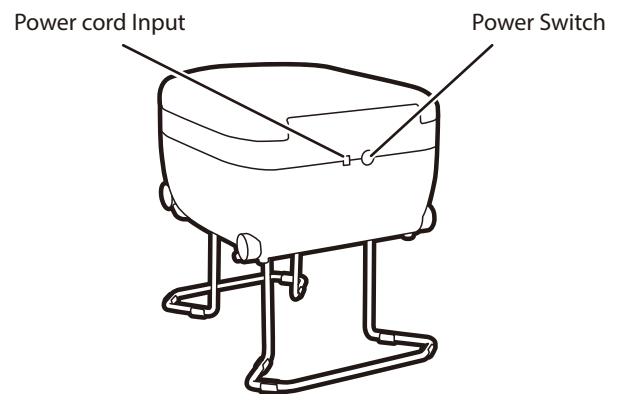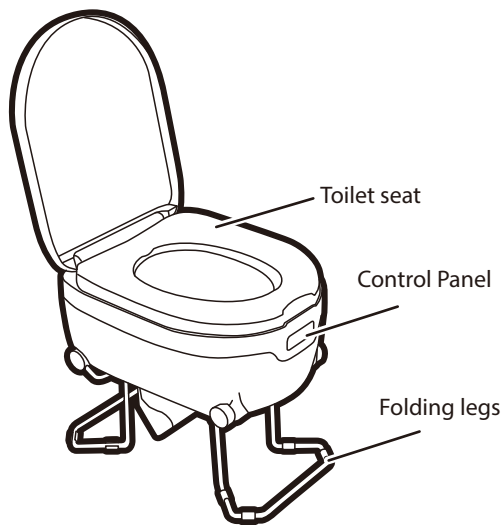

Top view with the toilet seat removed

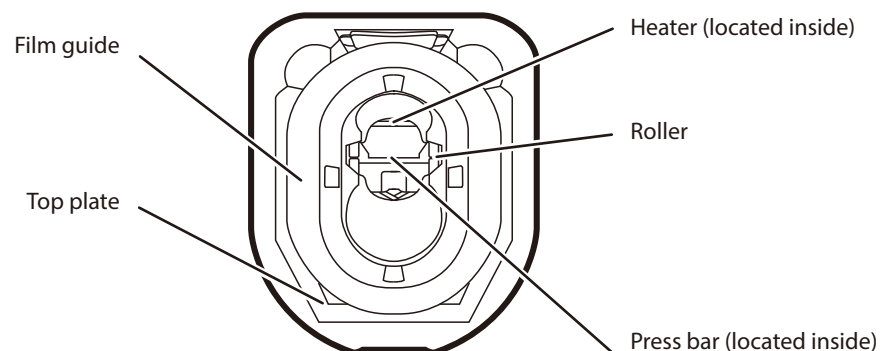

Control panel & Display

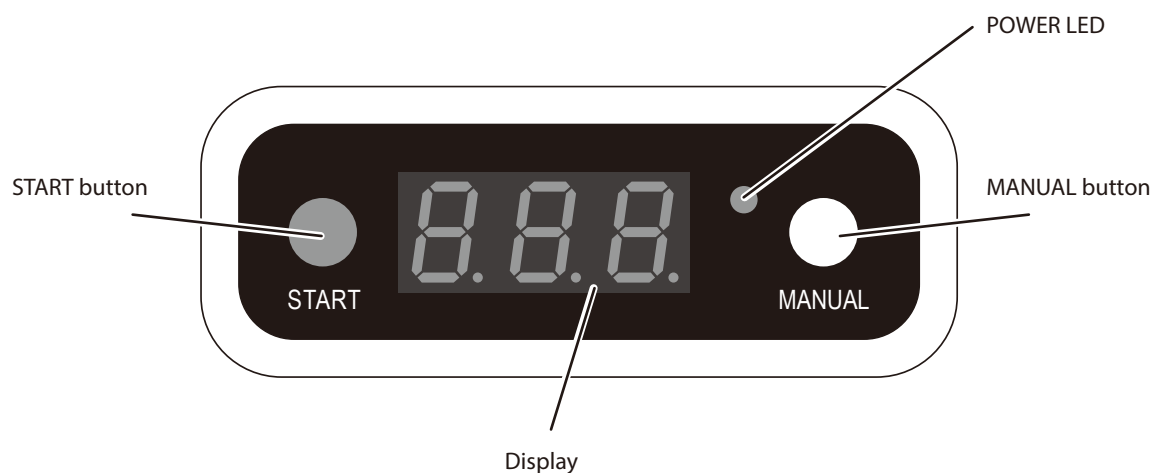

Description	Function
START button	Press and hold "Start" button for 2 seconds, and you will hear a beep soound, then the toilet starts sealing.
MANUAL button	Press and hold the "Manual" button to pull the liner further down, Use case1 : To create larger bags. Use case2 : To change the liner.
POWER LED	POWER lamp ON : When the switch on the back of the toilet is turned on. POWER lamp blinking : Sleep mode. POWER lamp OFF : When the switch on the back of the toilet is turned off.
Display	Operating mode / sealing mode : Countdown until the sealinng process is completed. The system will enter sleep mode if there is no operation within 80 seconds, and can be woken up by pressing any buttons on the control panel.


Main Unit Setup

- Read 'Ensure the safe use' on page 4~8 before assembling.
- Check 'Parts and accessories list' on page 9 to make sure all are available.

1

Expand the folding legs.


Lift the body and expand the folding legs until you hear a click.

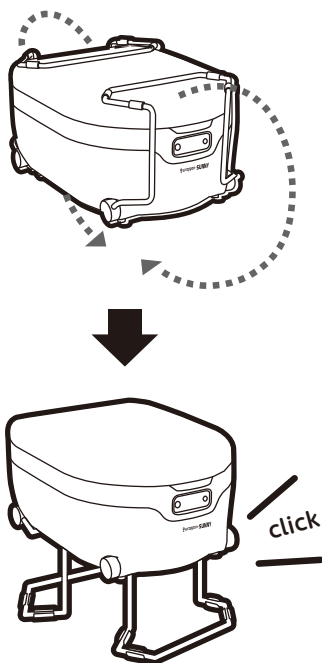

The guid plate is automatically open when the legs are positioned correctly.

2

Connect the AC adapter.

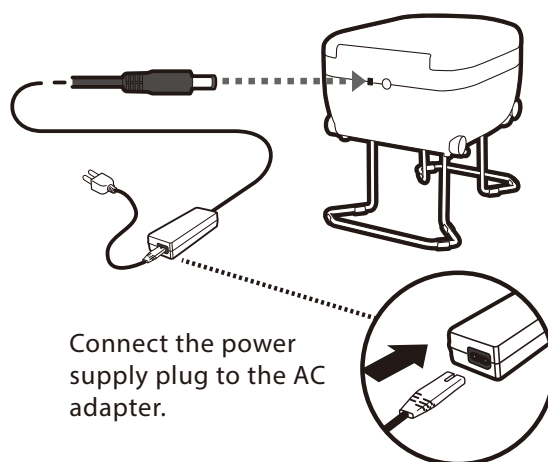

Connect the power supply plug to the AC adapter.

By this the assembling is finished.
Do not connect to the power supply before installing the film roll.

Installing a film roll

- Before installing a film roll, check 'main unit setup' on P12.
- While the plug is on, do not insert your hand inside the toilet.
- To function wrapping mechanism properly, a film roll must be installed properly.
- Do not use other than the designated film roll.




Please see the video "How to Install the Wrappon Sunny Film Roll" for details.

1

Make sure that the power supply is disconnected from the outlet.


Remove the toilet seat and film guide.

Unlock four clips and remove the film guide at upward.

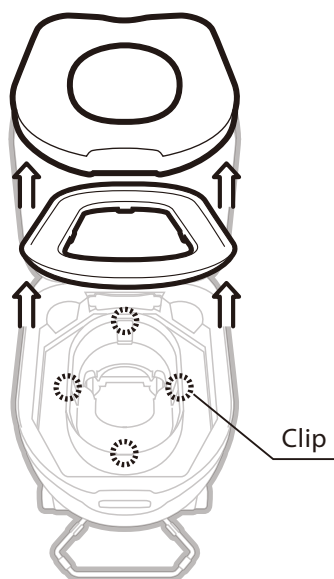


2

Install a film roll along with film guide.

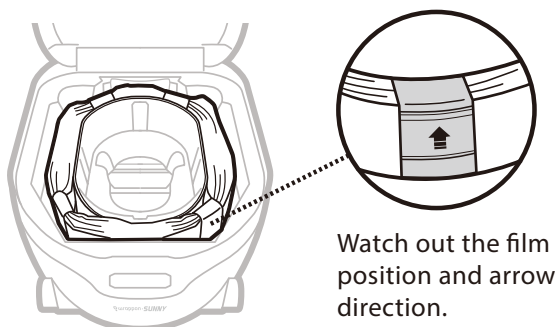

3


Remove Clamping bands at four locations of the film roll.

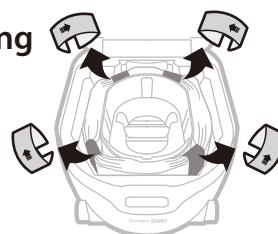

4

Install the film guide which you took at 1.


Push it until you hear the click sounds.

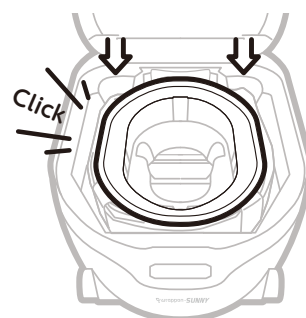

5

Take the film cassette cover out.


- ① Lift up the uppermost edge of the film roll from the upper end of the film guide by approximately 30cm.

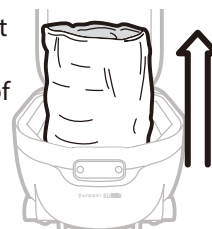

- ② Grip the upper portion of the film roll in a lump.




- ③ Push the lump in the toilet as far as you can.

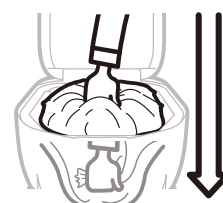


Installing a film roll (continued)


- 6** Remove the film installation jig from the holder.

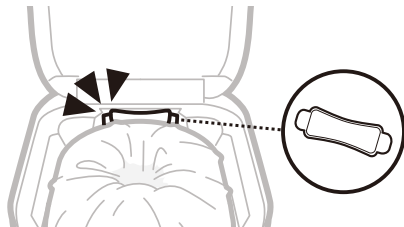

- 7** Push down the film installation jig while aligning it with the arrow on the main body.

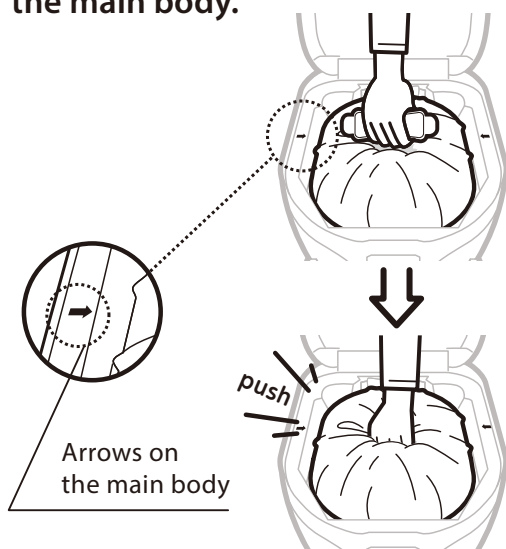

- 8** Pull up the film installation jig. Check if the film is properly set or not.

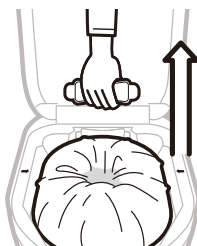


Film must be sandwiched both left & right set of rollers.




Film must be with one set of rollers.

- 9** Install the toilet seat.

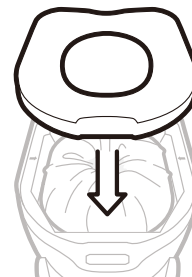


- 10** Put the power plug into an outlet. Press the power switch. Check that the display is illuminated.

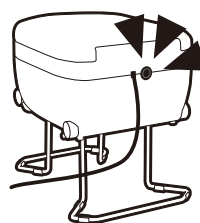

- 11** Press and hold Start button for 2 sec. To perform the initial sealing.

A beep will sound and the heat sealing will begin.


*Without this, the bottom of the bag is not sealed.

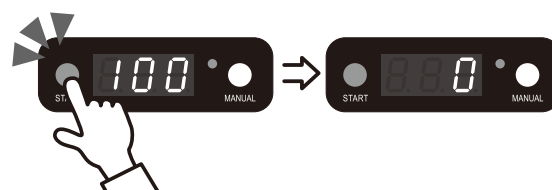

Display screen shows an estimated time remaining to complete the wrapping (100 countdown).

When completed wrapping, you hear beep sounds.

- 12** Take out the separated film. This completes film setup.

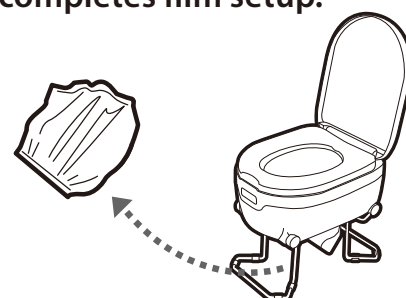

Changing the film roll


When you see the 'STOP' mark in the film



Please see the video "How to Replace the Wrappon Sunny Film Roll" for details.

1 Change the film roll immediately.

If you continue using the film, the film roll is running out, resulting a cause of waste leak.

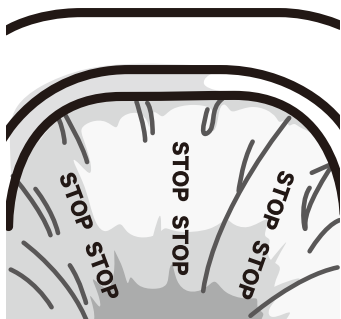


2 Press MANUAL button.

By holding down the MANUAL button, the film is sent out .


You keep pulling the film roll below gently until the film is running out.

If you did not pull the film, it may cause the film wrapped around the rollers.

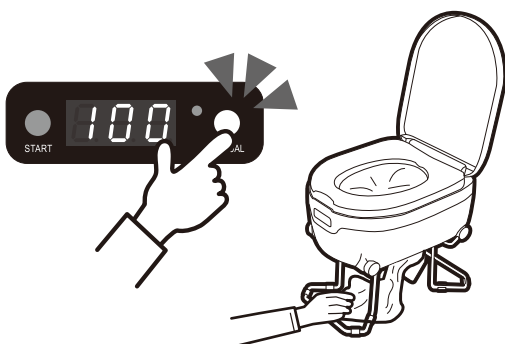


3 Turn off the switch on the back, then pull the electric power cord from the Outlet.

4 Take out the toilet seat .

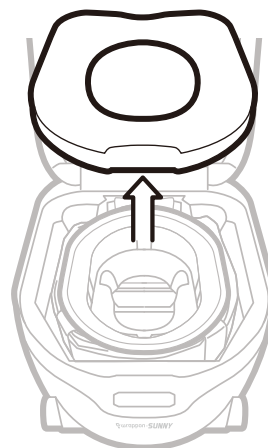

5 Please apply the included oil to the surface of the heater.

*If you experience film sticking after long-term use, applying the oil can help improve the condition.

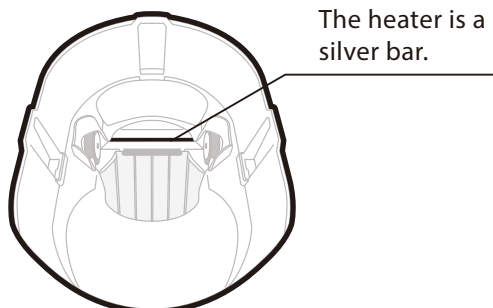

6 Install a film roll.

Refer to P.13-14, steps 2 to 12 of "Installing the Film Roll."

How to use the toilet

- Before using, make sure that the plug is connected to electric power outlet.
- Do not insert hand into the toilet.
- Sit down on the toilet as Western-style toilet.
- Do not stand and urinate. You should sit down on the toilet and urinate.
- Do not throw cigarette and sharp-edged items in the toilet. You can throw human waste, wet tissue and sanitary goods.
- If you drop something in the middle of wrapping, wait until the cycle finishes. Then, you open the wrapped film and take it out.
- There is a possibility of waste leaking as the wrapped film is not 100% sealed due to the environment and condition. Also make sure that you use the coagulant and the tray.


★ Check before use ★

- Make sure that POWER light ON.
- Make sure that you check if the film roll is installed. If the STOP mark is appeared on the film, replace it with a new film roll(refer P15).
- Check if there is no damage to the part of toilet seat that touches the body. If there is a damage, please do not use it.



Please see the video "How to Use Wrappon Sunny" for details.

- 1** Pour one packet of special coagulant. Before you do defecation, make sure that you pour the coagulant.

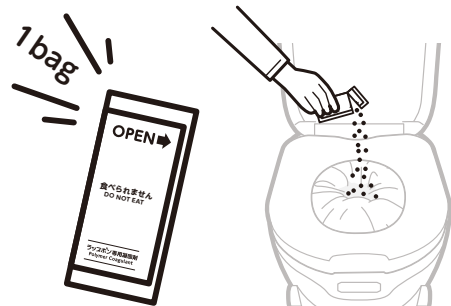

- 2** Relief yourself. Sit on the soft toilet seat in the correct position.


Push down the toilet paper so it is not sandwiched with press bar.

- 3** When finished, stand up and press and hold the START button for two second.


Make sure that you are away from the toilet seat.

After 100 countdown, you hear Beep sounds.
The wrapped film is automatically cut.

Do not pull the film until the wrapping is completely finished.

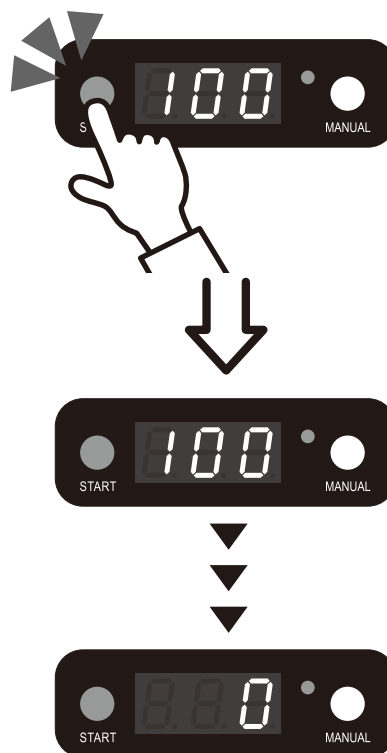

- 4** **Discard the wrapped film**
Disposal of wrapped waste film must follow the local government.

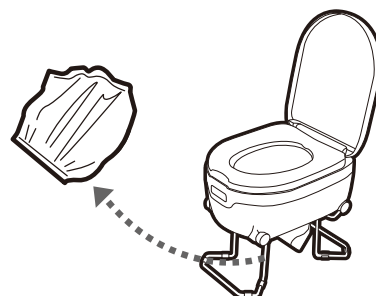

Maintaining the Product

Maintaining the product regularly to ensure hygienic use over a long period.


- Before performing maintenance or inspection, disconnect the power supply plug.
- Regularly dust the power supply cord.
- Keep the product clean.
- When cleaning, never use toilet cleaner, benzene, solvent or cleanser.
- After you finish cleaning, reinstall the toilet seat.

Daily maintenance


- For light soiling, wipe with a wet soft cloth lightly.
- For heavy soiling, wipe with a soft cloth soaked with diluted mild kitchen detergent.

Inside toilet is soiled

If the inside toilet is contaminated with human waste, do not insert hand into toilet. Stop the operation immediately. Pull off the power cord from power source. Please call the dealer you purchased.

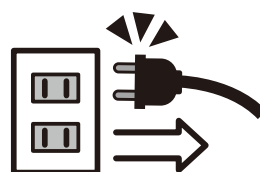


 **Warning**
Power cord off

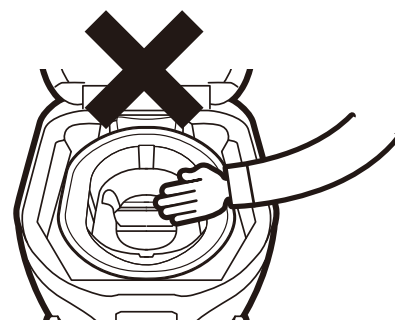

 **Warning**
Do not put your hand

Installing the film roll

When you change a film roll, it is recommended to wipe the back side of toilet seat and the top plate.

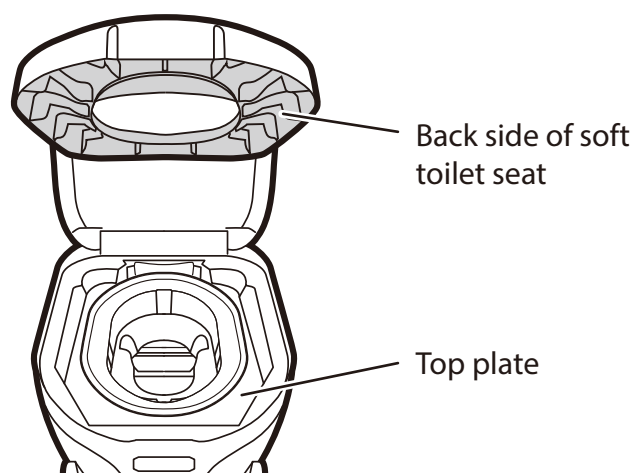

Storing the product

- Before storing the product, clean the toilet seat and other parts by referring 'Maintaining the product'.
- When the product is not used immediately, disconnect the power supply plug from the electric power outlet.
- If wrapped film is stored for a long time with waste inside, odor and smell occur. Earlier disposal of the wrapped film is recommended.


About the storage environment

An ideal storage location is an indoor with temperature between 0°C~40°C with low humidity.

Caution

Do not use the location as listed below.

- Dusty location
- Locations such as bath room where the humidity exceeds 80%.
- Locations exposed to rain or direct sunlight.

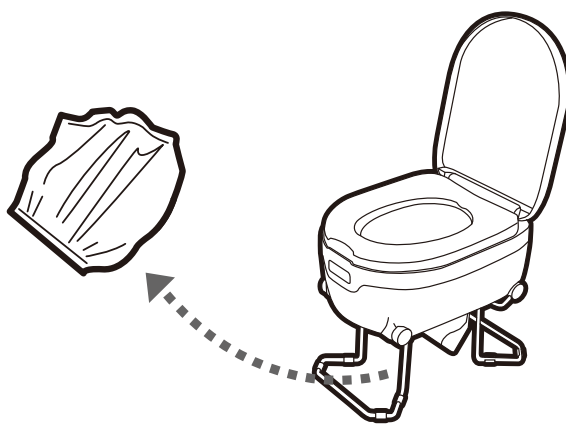

Frequently asked questions

Question	Answer
Will the wrapped waste not emit odor even if left as is.	The wrapped film does not prevent odors for an extended, so please of it promptly after use.
Isn't replacing the film roll difficult?	The film roll is designed to change without difficulty. Just follow the film roll replacement section in this manual or quick reference.
Can I dispose of the wrapped waste as combustible waste?	Like disposable diapers, wrapped waste is treated as combustible waste. However,, because the specific rules vary among individual local governments so check with your local government.
How should I clean the product?	Gently wipe with a soft cloth soaked with diluted mild kitchen detergent and rinse with a damp cloth dipped in water and rung out. The unit contains a heater for wrapping the film. Therefore, never pour or spray water on the unit when cleaning the unit. Otherwise, the heater may malfunction.
Why do I have to use the wet tissue?	Toilet paper can also be used. If light paper such as toilet paper is used, it may become trapped during the wrapping operation and cause waste leakage. Therefore, we recommend using wet tissue.
Is it OK to flush waste that has been coagulated with the polymer coagulant down a regular toilet ?	No. Because the polymer is water-insoluble, do not flush it down a regular flushing toilet.

Error Codes

Error	Fault	Solutions
E1	Thermosensor protection	Contact your dealer.
E2	Heating device protection	Contact your dealer.
E3	Timer protection There is foreign object in the clamping position.	Remove object and restart it by switching off and on. If the fault persists, please contact your dealer.
E4	Overcurrent protection when clamping	Remove object and restart it by switching off and on. If the fault persists, please contact your dealer.
E5	Overcurrent protection when declamping	Remove object and restart it by switching off and on. If the fault persists, please contact your dealer.

Troubleshooting

If the product does not work correctly after you have tried the steps described in the table below, contact the seller.

When this happens		Check the following	What to do
Wrappoon Sunny status	The control does not light.	Is the power turned off ?	⇒ Please turn on the switch located at the back of the toilet.
		Is the toilet in sleep mode ?	⇒ Press any button on the control panel to restart .
		Is the power cord or plug disconnected ?	⇒ Ensure the power cord and plug are firmly connected to the unit and the outlet.
		All other cases	⇒ Please contact your dealer.
	Unstable	Are the legs fully opened ?	⇒ Open legs till you hear a click.
During in use	Used without installing the film roll.		⇒ Immediately stop use and unplug the power cord. Contact your dealer afterward.
	If the film in the toilet displays a red "STOP" mark.		⇒ This indicates that the film roll needs to be replaced. Please refer to page 15 for "Replacing the Film Roll.
	Power Outage During Wrapping		⇒ After power is restored, press the Manual button to advance the film by about 10 cm. Carefully cut the film near the shooter to prevent spills. Tie the unsealed side and dispose of it. Press the Start button to resume heat sealing.
	If power is lost during wrapping.		⇒ The wrapping process will resume once power is restored. Do not pull the wrap forcibly during use.
	If you smell burning or see smoke		⇒ Immediately stop use and unplug the power cord. Contact your dealer afterward.
	If a coagulating agent is ingested		⇒ Immediately remove it, rinse your mouth thoroughly with water, and seek medical advice.
	Wrapped film is not cut away.	Look into the toilet to see if wet tissue is stuck in it.	⇒ Push and hold START bottom on the control panel for 2 seconds(one cycle length is rolled with empty inside). Follow the procedure of the Re-Wrapping on page 26.
		Is the film stuck to the heater ?	⇒ Please remove the film from the heater and apply the included oil to the surface of the heater using the provided cloth. (If the cloth is not available, you may use a dried paper towel or other fabric.)
		All other cases	⇒ Please contact your dealer.

When this happens		Check the following	What to do
During in use	Waste has leaked.	Did you forget to press the flush button after changing the film?	⇒ Follow "Correct re-wrapping"
		Is there a hole in the wrapped film?	⇒ Follow "Correct re-wrapping". If you press the button while seated, this may cause the film to tear. You must press the START button after standing up.
		Did you pull the wrapped film forcefully before the film is cut?	⇒ Follow "Correct re-wrapping".
		Did you forget to pour the polymer coagulant in?	⇒ This product does not seal completely. Depending on the operating condition and condition such as wrong film setting, the waste inside the film may leak so make sure you pour polymer coagulant. After that, refer to "Correct re-wrapping".
		All other cases	⇒ Call the dealer.
	Waste is not wrapped correctly.	Has the film come off the rollers?	⇒ Follow the steps under 'Installing the film roll' on, use the installation jig to insert the film between the rollers.
		All other cases	⇒ Call the seller

Re-wrapping

- 1 Open the film that has been incorrectly wrapped.
- 2 Firstly, pour a scoopful of coagulant. Then, put the film of 1.
- 3 Press and hold the START button of remote control unit for two seconds.
 - If Wrappon Sunny becomes soiled with waste, refer to 'Maintaining the Product' on page 19.
 - Should the waste get on your hand, wash it thoroughly with water and disinfecting soap, and then, disinfect with alcohol.

How to take out the film while in use


· Before working, please unplug the power plug from the outlet.

1 Prepare scissors




2 Lift the outside of the film.

Lift up to a height of about 40cm from the top of the film guide.

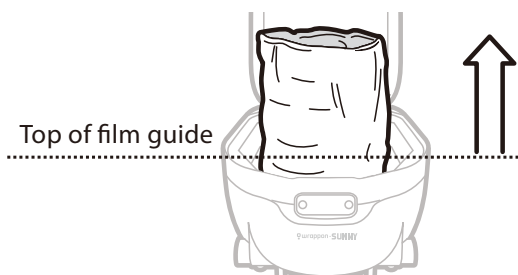



4 If the contents are inside the film, make sure to tie the inside film tightly to prevent the contents from spilling.

Please reduce the amount of air inside as much as possible.

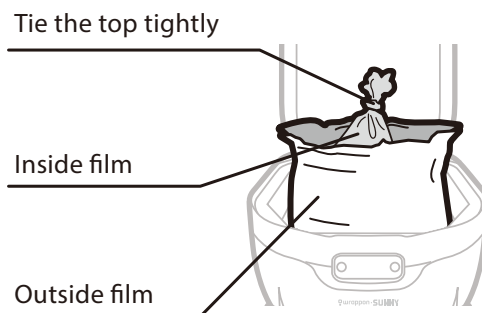


3 Cut the top of film with scissors.

The film is divided into two parts, inside and outside.

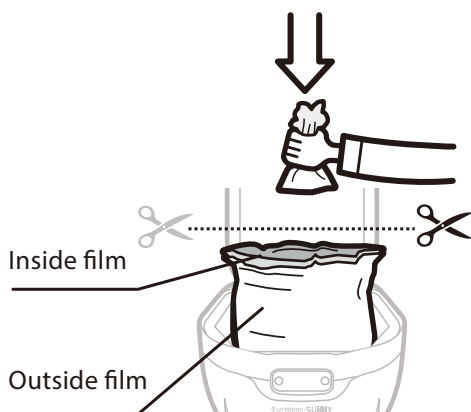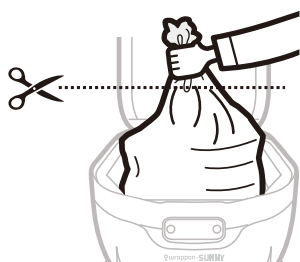

5 Pull out the inside and remove it from the roller.

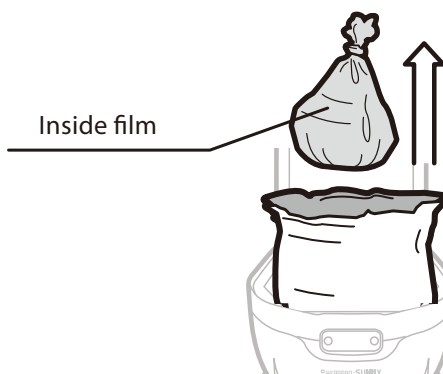

6 When installing a film, please refer to item 5 of "Install the film roll" on page 13.

Specification

Item		Specification
Product Name		Auto-wrapping(tear-and-seal) portable toilet Wrappon Sunny
Model No.		SNSE0002E
Materials		Stainless steel, ABS, PP, POM
Power supply	Input voltage	AC100V-240V 50-60H(AC adapter) or DC12V
Power consumption		60W(MAX)
Dimensions	Assembled dimension	36cm(Length) x 46.9cm(Depth) x 42.1cm(Hight)
	Folded dimension	36cm(Length) x 46.5cm(Depth) x 24.7cm(Hight)
	Height to the toilet seat	41.6cm
Weight		Main body: Approx. 7kg, AC adapter Approx. 0.5kg
Maximum load capacity		Less than 150kg
Ambient temperature	When operating	0°C~+40°C
	When not operating	-20°C~+60°C
Ambient humidity	When operating	20~80% (No condensation)
	When not operating	5~95% (No condensation)
Ambient atmosphere		Corrosive gas must not be present.
Safety function		Pinch protection*1

*1 Pinch protection is the function to stop the movement when noticed the foreign material other than film during sealing movement. Do not put your hand or body into the toilet.

These specifications may be modified without prior notice.

Wrappon Product Warranty & Free Replacement Policy

The warranty is valid for one year from the purchase date.

1. If the product malfunctions under normal use as per the user manual during the warranty period, we will provide free maintenance support. If a replacement is necessary after inspection, we will replace the product free of charge.
2. For maintenance support, please contact the dealer where you purchased the product.
3. The warranty does not cover:
 - Damage from improper handling, transportation, or dropping after purchase.
 - Damage from fire, natural disasters, or abnormal voltage.
 - Issues caused by external factors unrelated to the product.
 - Unauthorized repairs or modifications.
 - Improper use not following the user manual.
 - Damage or contamination of accessories and consumables.
 - Issues caused by incorrect power supply or fluctuations.
4. For maintenance inspection, if the product is deemed eligible for free replacement, the original product will not be returned to the customer.

***This warranty is only valid within the United States and Canada.**

[illegible]



■ Manufacturer

 **NIHON SAFETY CO.,LTD.**

<http://www.wrappon.com>

Wrappon Division,Nihon Safety Co.,Ltd.
11F,Ichibancho Tokyu Bldg.
21 Ichibancho,Chiyoda-ku,Tokyo 102-0082 JAPAN
Phone : +81-3-6369-2223
E-mail:wrappon@nihonsafety.com
 is a registered trademark of NIHONSAFETY CO.,LTD.

■ Agent
